

Please Read Carefully:

NOTE: THE MEDICAL PROFESSIONAL IS RESPONSIBLE FOR PROPER SURGICAL PROCEDURES AND TECHNIQUES. THE FOLLOWING INSTRUCTIONS ARE FOR INFORMATION ONLY. THE ATTENDING SURGEON MUST EVALUATE THE APPROPRIATENESS OF THE PROCEDURE BASED ON HIS/HER OWN MEDICAL TRAINING AND EXPERIENCE

Indications:

The Swiss Therapy Eye Mask was designed as a lightweight, cooling, non-adherent, eye compress to be used postoperatively following any surgical procedure around the eye zone requiring cold therapy.

How to use The Swiss Therapy Eye Mask

- Step 1. Prior to application, the tray containing the masks should be refrigerated for 10-15 minutes. Masks may also be cooled by placing the tray, with the mask, on a bed of crushed ice. **DO NOT FREEZE**
- Step 2. To remove the sterile mask from its tray
- Remove the mask tray from its outer wrapper. Note **the clear plastic sheet** covering the mask in the tray.
 - Grasp the plastic sheet **and** the mask's tab.
 - Gently pull the sheet/mask, in a parallel direction, removing the mask from its tray
 - Grasping the tab(s) place over the eyes and nose area.
 - Gently separate the sheet from the mask
- Step 3. One mask should be placed over the nose and eye area immediately post-op. A second mask in its tray should be kept cooling next to the patient on a bed of crushed ice. The third Swiss Eye mask should be given to the patient to be used at home.
- Step 4. To provide continuous cooling for the patient, the masks should be rotated every 15-20 minutes.
Note: The patient may, at first, experience the sensation that the mask is no longer cooling. This sensation is due to the skin being desensitized by the initial cold.
- Step 5. At home: After each use the mask should be rinsed with a sterile balanced salt solution (contact solution). The mask should then be stored in the refrigerator in their trays, inside a zip lock bag.
- Notes:
- Each mask is for single patient use.
 - Do not freeze
 - To prevent possible future infection of the eye, masks should not be used beyond the postoperative period suggested by your physician.

Warranty


Invotec International, Inc. warrants that the product is free from defect in material and workmanship. Invotec will replace or provide a refund for any product found to be defective so long as the product is returned according to the Returned Goods instructions in the Sales Policy. Invotec shall not be liable for any consequential loss, damage or expense directly or indirectly arising from the use of or inability to use this product. THE FOREGOING WARRANTY IS IN LIEU OF AND EXCLUDES ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, HOWEVER ARISING, INCLUDING MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, AGAINST INFRINGEMENT OR OTHERWISE.

Invotec International neither assumes, nor authorizes any person to assume for it, any other additional liability or responsibility with respect to this product.

The available range of Swiss Therapy range products

5x9 cm, 7x12 cm, 10x10 cm, the eye mask 7x14cm, the mask on the face 25x25 cm, the eye pads 7,5x3,5 cm

Manufactured by:

 KIKGEL, ul. Sklodowskiej 7, 97-225 Ujazd, Poland

For:**INVOTEC INTERNATIONAL, INC.**

6833 Phillips Industrial Blvd. • Jacksonville, Florida 32256 U.S.A. • Tel: 800-998-8580/904-880-1229 • Fax: 904-886-9517

www.invotec.net

